
Instructions for use Contact™ Nipple Shield

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Notkunarleiðbeiningar Contact™ Nipple Shield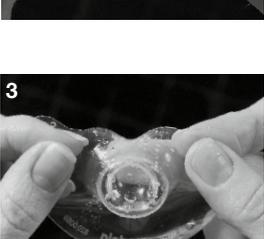

EN Contact™ Nipple Shield

Important Safeguards
<p>Read all instructions before use.</p> <ul style="list-style-type: none"><i>Use the product only for its intended use as described in this instruction manual.</i> <i>The contact nipple shield is a personal care item and is not to be shared between mothers without proper sterilization to prevent the risk of cross-contamination.</i> <i>We recommend seeking advice from a health professional or a lactation consultant.</i> <i>Inspect before each use and replace if damaged or worn out.</i> <p><i>SAVE THIS INSTRUCTION MANUAL.</i></p>

Instructions For Use
<ul style="list-style-type: none">Wash hands thoroughly with soap and water before touching the breast and shield. • Clean the breast and surrounding area with a damp cloth. Do not use soap or alcohol which can dry the skin and cause cracked nipples. • Moisten the shield with water as this will help it to stick to the skin. <p>Push the tip almost inside out (picture 1) and grip the wings of the shield (picture 2). Cover the holes with the middle finger and place the shield centrally over the nipple (picture 3). Keeping the finger over the holes, gently pull the wings of the shield until it fits snugly over the nipple (picture 4).</p> <p>Note: The open section should point in the direction of the baby's nose during feeding.</p>

Both mother and baby need to be considered when choosing the correct size nipple shield. Assess the size of the nipple, so the shield fits comfortably, without pinching and the nipple fills the cavity wide-wise. In the case of prematurely, a smaller size may be needed to ensure that the baby is able to grasp the shield so the lips reach the flat base, against the breast. Available in 3 sizes: S/16mm, M/20mm or L/24mm.

Cleaning and Hygiene
<p>Important</p> <ul style="list-style-type: none">Use drinking water quality for any of the following described cleaning procedures. Clean shields immediately after use to avoid dry up of milk residues and to prevent from growth of bacteria.

Cleaning at home

Follow these cleaning instructions unless you are told otherwise by your healthcare professional.

Prior to first use and once daily:

- Wash and rinse Contact Nipple Shields.
- Place shields in a saucapan of sufficient capacity.
- Fill with cold water until the shields are well covered
- Optionally, a tea spoon of citric acid may be added to avoid a build-up of limescale.
- Heat it up and allow to boil for 5 minutes.
- Remove from stove and drain water off.
- Dry shields with a clean towel.

Alternatively the shields can be

- cleaned in top rack of dishwasher and sterilized as follows
- using the Medela Quick Clean Microwave Bag or
- steam-sterilized according to manufacturer's instructions

Note: Food colorings may discolor components which will not affect performance.

After each use

- Handwash Nipple Shields in warm, soapy water.
- Rinse in clear water.
- Dry shields with a clean towel.

Cleaning in hospitals

These cleaning instructions are general recommendations and may individually be adjusted based on the hospital's specific directives and cleaning practices.

Important

It is recommended to provide each mother with own personal Nipple Shields.The nipple shield must be sterilised before being used for the first time. It must be cleaned, disinfected and sterilised (e.g. in an autoclave) after each use, or at the latest before it is given to the next mother. Use by more than one mother without adequate sterilization may pose a health risk (e.g. risk of cross-contamination) and voids the warranty.

Cleaning

Thoroughly wash Contact Nipple Shields in warm, soapy water. Rinse in clear water.

Disinfecting

Disinfect by washing in a thermic washing machine (at approx. 93°C).

Sterilizing

Prior sterilizing visually check shields if clean and intact. Sterilize all shields by autoclaving at 134°C for 3 (or 18) minutes or at 125°C for 15 – 20 minutes. Do not stack shields or apply external pressure to avoid damages.

Storage-Box

After cleaning, when Contact Nipple Shields are not in use, put in storage box.

Cleaning of storage-box at home

The storage-boxes can be

- handwashed in warm, soapy water or
- cleaned in top rack of dishwasher or
- santitized with the Medela Quick Clean Microwave Bag or
- steam-sterilized according to manufacturer's instructions

Cleaning of storage-box in hospitals

Thoroughly wash storage-box in warm, soapy water. Disinfect by washing in thermic washing machine (at approx. 93°C).

Disposal
<p>According to local regulations</p> <p>CE-mark according to 93/42/EEC</p>

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